# HUMEX

## SPRAY AURICULAIRE

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## EAR SPRAY

### UK EAR HYGIENE AND CLEANSING

This spray is intended for an hygienic gentle cleansing of the ear. The wall of this canal naturally secretes earwax, the yellow substance which protects the ear from dust, foreign bodies and infections. However, earwax can build up, forming plugs and reducing hearing. Regular use of HUMEX ear spray, instead of cotton buds, will prevent earwax from building up and forming plugs.

**INSTRUCTIONS FOR USE:** Never remove the transparent nozzle

1) Spray into each ear: two to three times per week as a general rule and once or twice daily in the event of significant exposure to dust, use of hearing aids.

2) Turn the head to one side and wipe residual liquid away.

3) Rinse the spray dispenser with hot water after use.

**DO NOT USE IN THE FOLLOWING CASES:** - history of eardrum damage - ear tube fitted - presence of a foreign body in the canal - ear infections - infants under 30 months of age - use of ear drops.

**PRECAUTIONS BEFORE USE:** Never remove the transparent nozzle. Do not store in the refrigerator. Do not insert the nozzle too far into the ear since this may cause irritation to the canal. Seek medical advice in the event of pain. Keep out of the reach of children. Only for use in ears.

**PRESSURIZED CONTAINER. PROTECT FROM SUNLIGHT AND DO NOT EXPOSE TO TEMPERATURES EXCEEDING 50°C. DO NOT PIERCE OR BURN, EVEN AFTER USE.**

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