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Physiological Sea water Microspray
A clear nose is the key to good breathing

The nose and the nasal mucosa have an important role. They humidify and warm up the air breathed in like an air conditioner. When it is too dry or congested, the nose can't function properly. Moreover, mucosities are often responsible for contaminating the ENT sphere as they carry pathogenic germs.

Thanks to its thin microdiffusion, **STÉRIMAR®** allows a decongesting and gentle cleansing of nasal fossae, avoiding projection of mucosities in the back of the throat. **STÉRIMAR®** brings an immediate respiratory comfort. More than a physiological serum, **STÉRIMAR®** provides marine trace-elements properties.

PROPERTIES:

STÉRIMAR®, medical device, is indicated for babies, children and adults in daily nasal hygiene.

The sea water is collected in deep waters over 7 miles from the coast line where it can't be soiled by coastal currents. After checking, the sea water is purified and made physiological.

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USE AND ADMINISTRATION:

STÉRIMAR® natural product, is indicated for daily nasal hygiene to:

- learn the reflex of blowing the nose and make it easier,
- restore the natural humidity of the nose,
- prepare and facilitate the action of a specific nasal treatment,
- clean the nose from secretions.

STÉRIMAR® can be use in every position. Its specific anatomical nozzle suits safely small nostrils of babies as well as adults ones.

1. Gently insert the nozzle into the nostril
2. Press the nozzle briefly.
If your nose is blocked or stuffy, carry out an irrigation by continuing to press the nozzle for a few seconds.

3. Let any excess solution run out – this should take the mucus with it. Then blow your nose.
4. Repeat the procedure for the other nostril.
5. Remove the nozzle and wash with soapy water, rinse and dry.

Use 2 to 6 doses per day in each nostril. **STÉRIMAR®**, natural product, without preservative, can be used on a long period.

COMPOSITION:

Sea water: 31.82 ml for 100 ml

04346029